MoVeS

Knee Rope PULLEY

on - Medical Device - Instructions for Use
Intended purpose: To increase lower body range of motion and improve strength. **Intended user:** Lay person and healthcare professional. **Indications:** For rehabilitation after lower body injury.

Contraindications:
 • Do not use in case of fractures;
 • Do not use in case of severe oedema.

Warnings:
 • Consult a healthcare professional before use;
 • Maximum load: 100kg (220 lbs).

Precautions:
 • In case of unusual swelling, skin discoloration or any other discomfort, stop using the device and consult a healthcare professional.
 • The door to which the device is attached should be of standard thickness, closed and locked before using the device.

• Device should not be attached to a glass door;
 • Always examine the Knee Rope Pulley before use for cuts, small tears or punctures. If you find any flaws, do not use device.

Care and cleaning: Clean with lukewarm water and mild soap and let dry.

Component:
 • 1 x door anchor strap with pulley system;
 • 1 x 2-handed wooden handle attached to nylon rope;
 • 1 x ankle strap.

General Instructions:
 1. Be seated during use;
 2. Use standard-width non-glass door to attach door anchor;
 3. Attach strap: Place strap between door and doorpost. Close door firmly and lock it. Unfold rope. Pull on rope to ensure door strap is properly in place and door is firmly locked;
 4. Pull your foot carefully upwards by pulling the handle with both hands towards your abdomen to generate a rotational movement on your knee.

Report any serious incident (death, serious deterioration of health, serious public health threat) to the Manufacturer and your Competent Authority.

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MoVeS

Knee Rope PULLEY

For exercises and more products
WWW.MOVES-YOU.COM

The MoVeS Knee Rope Pulley is an economical, easy-to-use device that helps increase range of knee motion. It can be used for prevention and rehabilitation of knee injuries.

This Knee Rope Pulley set can be mounted in a doorway or another mounting. Includes ankle support, nylon cord , pulley, door anchor and wooden 2-handed handle.

UK CA UK REP
 Qarad UK Ltd.
 8 Northumberland Ave
 Westminster, London WC2N 5BY
 United Kingdom

CE MD QTY 1


MoVeS

Knee Rope PULLEY

MoVeS

Knee Rope PULLEY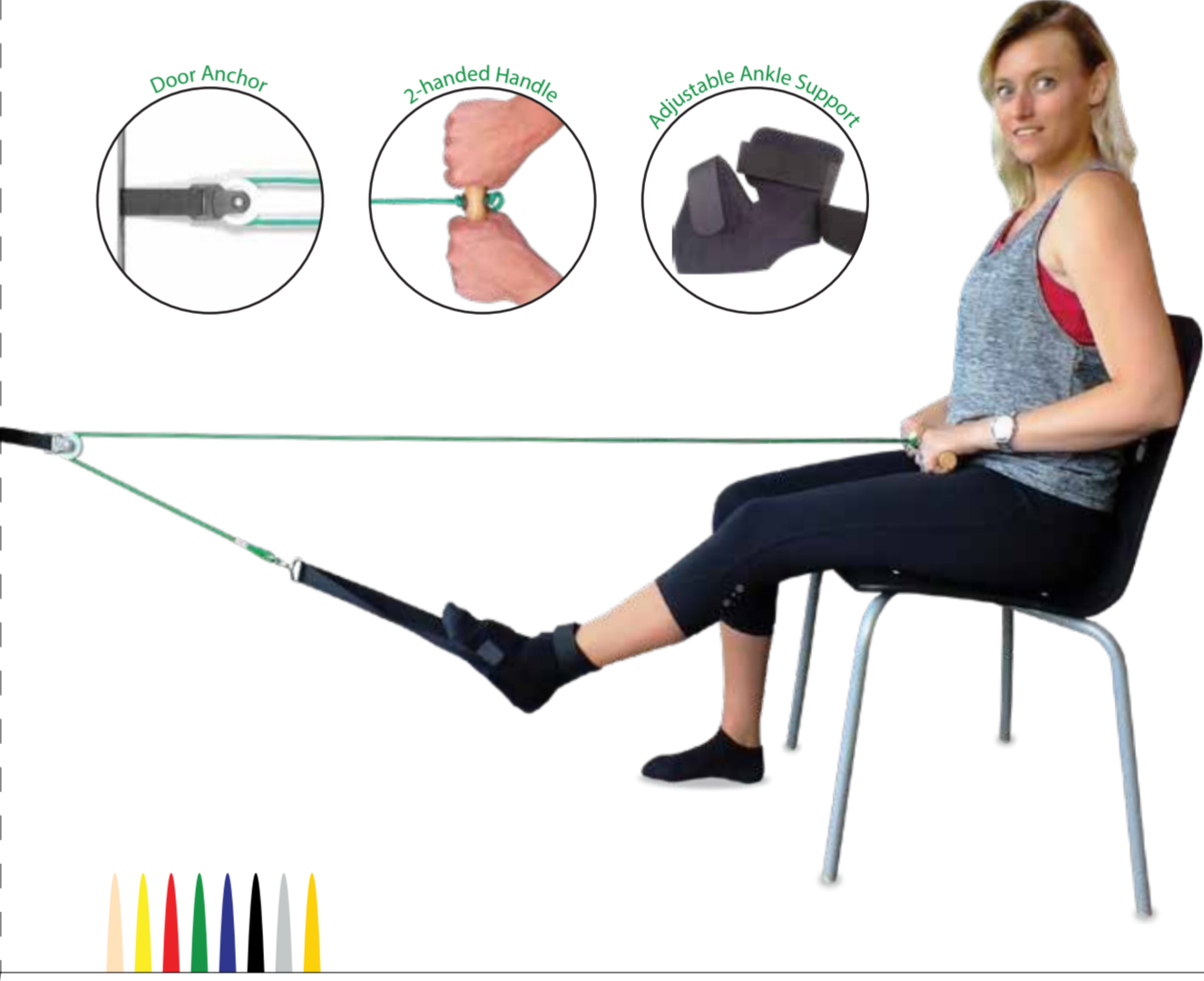

MoVeS

Knee Rope PULLEY

For exercises and more products
WWW.MOVES-YOU.COM

The MoVeS Knee Rope Pulley is an economical, easy-to-use device that helps increase range of knee motion. It can be used for prevention and rehabilitation of knee injuries.

This Knee Rope Pulley set can be mounted in a doorway or another mounting. Includes ankle support, nylon cord , pulley, door anchor and wooden 2-handed handle.

UK CA UK REP
 Qarad UK Ltd.
 8 Northumberland Ave
 Westminster, London WC2N 5BY
 United Kingdom

CE MD QTY 1

MVS
 IN MOTION
 MVS In Motion
 Westdijk 150
 2830 Tisseel (Willebroek)
 Belgium - Europe
www.mvs-in-motion.com

Made in Taiwan

REF 01-400301

UDI-DI 5 420063 003636