# MoVeS

Instructions for use  
en | fr | de | nl | es | it | pt | da | sv | fi | pl | el | tr

## ROLL-ON PACKS

07-050101  
Finger

07-050102  
Small

07-050103  
Medium

07-050104  
Large

07-050105  
Extra Large

## PEARL PACKS

07-041101  
Eye Mask

07-040102  
Sport Pack  
with Strap

07-040201  
Back Wrap

07-040301  
Knee/Elbow  
Wrap

Made in China

MVS In Motion  
Westdijk 150  
2830 Tiselt (Willebroek)  
Belgium - Europe  
[www.mvs-in-motion.com](http://www.mvs-in-motion.com)  
Find us on &

Qarad UK Ltd,  
8 Northumberland Ave  
Westminster, London WC2N 5BY  
United Kingdom

## MICROWAVE HEATING

## PRODUCT REFERENCE DESCRIPTION

REF	Power	(A) Standard Heating Time	(B) Unit Increments	(C) Max Total Heating Time
07-041101	700W	15s	5s	30s
07-040102	700W	25s	5s	40s
07-040201	700W	35s	5s	50s
07-040301	700W	35s	5s	50s
				
07-050101				
07-050102	750W	1min	15s	1min45s
07-050103	750W	1min	15s	1min45s
07-050104	750W	1min	15s	1min45s
07-050105	750W	1min	15s	1min45s

Remark: Start room temperature 20°-24°C

700 - 750W

75°C / 170°F

-18°C / 0°F

## **en – Medical Device – Instructions for Use**

Intended use: Application of cold or heat therapy to affected area.

Intended user: lay person and healthcare professional

Indications: All conditions where hot therapy helps muscles to relax by promoting blood flow, or cold therapy to decrease swelling by reducing blood flow. Alternating heat and cold may help reduce exercise-induced muscle pain.

Contra-indications for hot or cold therapy: Individuals who are sensitive to temperature extremes, such as those with cardiovascular problems (e.g. deep vein thrombosis, raynaud's disease), diabetes, nerve damage, paralysis, skin irritation. If skin is broken, treat wound and apply (sterile) bandage to the injury before using.

Precautions & Warnings: Consult a healthcare professional before use.

Do not open the pack. Do not puncture the pack and/or ingest content. If contents come into contact with eyes or skin: flush with water and contact a healthcare professional. Never sit, lay or lean against the pack. Do not fall asleep while using the pack. Do not use along with sports creams, balms or lotions. Do not apply on sensitive skin areas. Always check to make sure pack is not too hot or too cold before applying. Use the Pack in a soft cloth or over clothing to avoid direct skin contact; for Soft Touch and Pearl Pack this is not mandatory but highly recommended. Apply for maximum 20 minutes. Wait at least 20 minutes before re-applying. An overheated pack may cause injury. Overheating may cause the pack to rupture. Do not use on infants.

How to use: Reusable. For external use only. For heating times: see table. Test the product on the body for 2 minutes. If a hotter Pack is desired, microwave it in (B) seconds increments until the optimum temperature is reached. DO NOT EXCEED THE TOTAL HEATING TIME INDICATED (C). Do not use in hydrocollator. For freezing times: see table. For longer product life: use for Hot or Cold, not for both. When placing the Pack in the freezer, lay it flat, not folded.

Report any serious incident (death, serious deterioration of health, serious public health threat) to the Manufacturer and your Competent Authority.

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Copyright © All rights reserved