MoVeS

120cm TUBE with Handles

Using the MoVeS Tube with Handles for resistive exercise provides positive as well as negative force on the muscles and improves range of motion, strength and cooperation of the muscle groups. The different resistance levels are determined by the thickness of the tube.

The MoVeS Tube with Handles increases in resistance from yellow to silver.

This pack contains MoVeS Tube of 120cm and 1 pair of interchangeable handles.

Colour	Resistance	REF	UDI-DI
Yellow	Light	01-200102	05420063000413
Red	Medium	01-200103	05420063000420
Green	Heavy	01-200104	05420063000437
Blue	Extra Heavy	01-200105	05420063000444
Black	Special Heavy	01-200106	05420063000451
Silver	Super Heavy	01-200107	05420063000468

For exercises and more products

WWW.MOVES-YOU.COM


 AUV3
MVS In Motion
Westdijk 150
2830 Tisselt (Willebroek)
Belgium - Europe
www.mvs-in-motion.com
Find us on  & 

UK
CA UK REP
Qarad UK Ltd.
8 Northumberland Ave
Westminster, London WC2N 5BY
United Kingdom

Made in Malaysia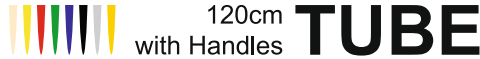

en – Medical device – Instructions for use
Intended purpose: To tone, shape, strengthen or rehabilitate muscle groups through resistance exercise, for use in physical therapy or fitness training. **Intended user:** Lay person or healthcare professional. **Indications:** Medical conditions resulting in limitation in movements, postural imbalance, muscle performance impairments. **Contraindications:** Use should be discontinued in case of unusual swelling, discomfort, dizziness, severe muscle or joint pain or pain in the chest. **Precautions:** - Consult a healthcare professional before starting an exercise program. Always examine MoVeS (FIT) Tube before use for cuts, small tears or punctures. Do not use if damaged; Store away from sharp objects. Remove rings and be aware of sharp fingernails; Store away from sunlight. **Warnings:** This device contains natural rubber latex which may cause allergic reactions; Do not stretch beyond three times the original length. Report any serious incident (death, serious deterioration of health, serious public health threat) to the Manufacturer and your Competent Authority.

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