MoVeS

KINESIOLOGY TAPE

REF

TAN 04-060101
 BLACK 04-060102
 PINK 04-060103
 LIGHT BLUE 04-060104

UDI-DI

05420063005555
 05420063005562
 05420063005579
 05420063005586

UK REF

Qarad UK Ltd,
 8 Northumberland Ave
 Westminster, London WC2N 5BY
 United Kingdom

MVS In Motion
 Westdijk 150
 2830 Tisselt (Willebroek)
 Belgium - Europe
 www.mvs-in-motion.com
 Find us on

WWW.MOVES-YOU.COM

en – Medical device – Instructions for Use

Intended purpose: To provide support and stability for joint, muscles and ligaments. **Intended user:** Lay person and healthcare professional. **Indications:** Medical conditions resulting in loss of strength, flexibility and coordination of movement.

Contra-indications:

- In case of unusual swelling, skin irritation, allergic reaction or discomfort, use should be discontinued;
- Do not use in case of pregnancy;
- Do not apply on open wounds or sensitive skin.

Precautions:

- Consult a healthcare professional before use;
- Apply at room temperature or slightly warmer;
- Do not apply for longer than 5 consecutive days, do not stretch the tape too much;
- It is advisable to apply directly on the skin;
- It is advisable to shave off body hair before applying MoVeS Kinesiology Tape;
- MoVeS Kinesiology Tape is not a first aid bandage;
- Avoid high temperatures & moisture while storing the MoVeS Kinesiology Tape.

General information: Latex-free; water resistant; single use.

Report any serious incident (death, serious deterioration of health, serious public health threat) to the Manufacturer and your Competent Authority.

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